Instructions for Use

PEDITROL

Irrigation System

Wismed CC
71 Princess Alice Avenue,
Durban 4001, South Africa
Tel: +27 31 206 0177
orders@wismed.co.za

TNMC Devices (Llc)
Ulica Franje Krežme 1
10000 Zagreb, Croatia
Tel: +38 59 1622 9658
tnmchr@tnmc.biz

Instructions for Use (EN.)	2
Mode d'emploi (FR.)	7
Gebrauchsanweisung (DE.)	12
Istruzioni per l'uso (IT.)	17
Gebruiksaanwijzing (NL.)	22
Bruksanvisning (SE.)	27
Instrukcja Używania (PL.)	32
Instruções de utilização (ES.)	37
用説明書 (JP.)	42
Vejledning til brug (DAN.)	47
Instruções de utilização (PT.)	52
Kullanım Talimatı (TR)	57
Uputstva za upotrebu (HR.)	62
Symbols	67

Scan to watch the
instructional video

Instructions for Use

INDICATIONS

For use with urological and/or urological endoscopic procedures requiring irrigation. Irrigation helps to maintain a clear visual field by displacing blood, stone fragments and debris. Only sterile irrigation fluid should be used.

CONTRAINDICATIONS

None known

USERS

Physicians and other qualified medical personnel.

PATIENT TARGET GROUP

Adult patients requiring diagnostic and/or therapeutic endoscopy in the upper urinary tract.

PREPARATION FOR USE

Inspect the package for any mechanical damage.

Do not use if package is damaged

CAUTION

All necessary manipulations during preparation and during intervention must be carried out in sterile field.

CAUTION

The Peditrol irrigation system should be assembled and used only by physicians and other qualified medical personnel who are familiar with equipment normally used during urological endoscopic procedures.

POTENTIAL COMPLICATIONS

Any irrigation product may cause the following adverse effects:

- Excessive distension of urinary tract and hydronephrosis
- Extravasation of irrigating fluid and fornix rupture, flank pain
- Postoperative fever, urinary tract infection, sepsis and septic shock
- Transient haematuria, subcapsular haematoma

The patient should be routinely monitored in accordance with accepted standards for endoscopic procedures.

INTENDED USE AND DISPOSAL

The Peditrol irrigation system is intended to be used on an individual patient during a single urological procedure and then disposed according to the hospital's Medical Waste Handling Procedure. It is not intended to be reprocessed and used again. If the device is reused, the patient would be at risk of infection and other complications.

DEVICE DESCRIPTION

Peditrol is operated by foot, leaving the surgeon's hands free for stone manipulation. It consists of a robust Foot Pump (URO 002 or URO 003) and a sterile, single-use disposable unit (URO 041). The syringe is connected to the disposable unit and mounted into the foot pump.

CAUTION

Use with Foot Pump URO 002 or URO 003

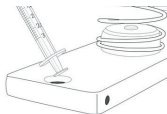
Scan to watch the instructional video

INSTRUCTIONS FOR USE

1. Connect the Luer Lock syringe to the Manifold tightly.

2. Insert the end of syringe plunger into bottom plate nest.

3. Without depressing the foot plate, pull the Manifold up and insert into its nest.

4. Attach the Female Luer Lock to the irrigation fluid supply.

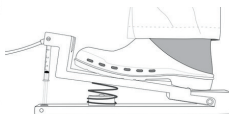
5. Ensure the blue In-Flow Switch is in the Open position.


Open ←→ Closed


6. Expel air from the tubing by depressing the foot pump several times.

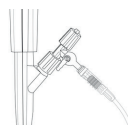7. Connect the In-Flow Switch to the ureterscope. Ensure all taps are open.

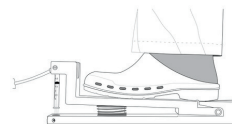8. To pump, depress the foot pedal to deliver the desired amount of fluid and release. The syringe will automatically refill.

FOOT PUMP

Instructions for Use

INDICATIONS

For the use with the Peditrol Irrigation System for urological and/or urological endoscopic procedures requiring irrigation.

CONTRAINDICATIONS

None known

POTENTIAL COMPLICATION

None known

CAUTION

The Peditrol Foot Pump should be used in conjunction with the disposable Peditrol irrigation set URO041 and should be assembled and used by suitably trained physician and/or other qualified medical personnel who are familiar with equipment normally used during urological endoscopic procedures.

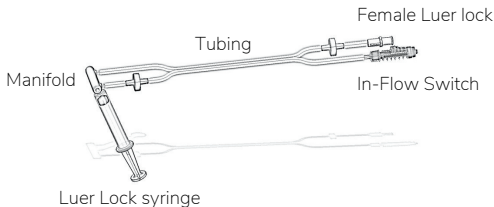
DEVICE DESCRIPTION

The reusable Peditrol Foot Pump (URO 002 or URO 003) is a driver for a disposable irrigation tubing set URO041. The disposable set is mounted into the pump and operated by foot.

HANDLING

The Foot Pump is a non-sterile accessory; and should be cleaned after every use according to the cleaning instructions to prolong the life span of the Foot Pump and to avoid the build-up a sticky residue from water, and bodily fluids such as blood or urine.

A. Assembled disposable unit - URO041

B. Disposable unit connected to foot pump

Cleaning Instructions After Use

REGULAR CLEANING

1. Rinse off the Foot Pump under warm running water.
2. Take a standard brush and apply detergent based cleaner and brush for +/- 5min concentrating on the joins and crevices.
3. Scrub the Foot Pump focusing on the:
 - Manifold nest (A)
 - Bottom plate nest (syringe insert) (B)
 - Spring (C)
 - Connection between the plate and heel rest (D)
 - Screws (E)
4. Rinse the unit thoroughly, using a soft bristle brush to remove any soap residue
5. Pat the unit dry, focusing on any areas where excess water may accumulate.
6. Once the unit is 100% dry, store in a cool, dry place.
7. Once a month spray the bearings, screws, spring and the plate and heel rest with a light duty lubricating agent.

TROUBLE SHOOTING

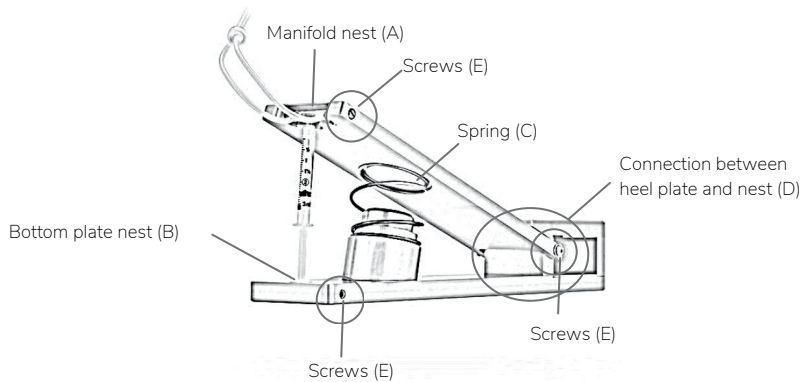
Should it become difficult to load the syringe into the nest, it may be due to a residue build up around the bearings inside the Foot Pump nest. This causes the small spaces around the pump to become sticky resulting in them being unable to move freely.

TROUBLE SHOOTING

1. Place the Foot Pump submerged in Warm – Hot water with a detergent based cleaner and soak overnight to allow the trapped dirt to become loose.
2. Follow the Regular Cleaning Instruction - Points 2 to 4
3. Take a large screwdriver and push the bearings in the nest back. You will need to apply a moderate amount of pressure. Once the bearing has been pushed back the spring will be released and the dirt trapped inside the pump nest can be removed, rinse under lukewarm water to remove excess dirt and use the soft bristle brush to assist in removing any excess dirt left behind.
4. Pat the unit dry, focusing on any areas where excess water may accumulate
5. Spray the unit with a 70%+ alcohol spray to assist the process of drying.
6. Once the unit is 100% dry, spray the bearings, screws, spring and the plate and heel rest with a light duty lubricating agent.
7. Store in a cool, dry place

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AB Yetkili Temsilcisi